



This guide is for Android 5+ Smartphone users.



AbiBird®


# How to Pause Sensor Notifications

You can pause a sensor's notifications – perhaps your cared-for person is away for the weekend.

To pause a sensor's notifications, you need to:



1. Access sensor graph page
2. Pause notifications

## Step 1: Access sensor graph page

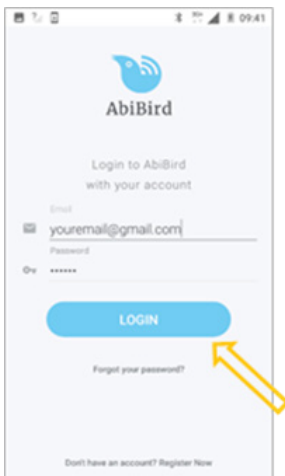

Login to AbiBird with your username (email) and password.  
The *My Sensors* page will open.

**NOTE:** If you are already in the AbiBird app you can access the *My Sensors* page by selecting the **Back Arrow** (top left) repeatedly until the *My Sensors* page opens.

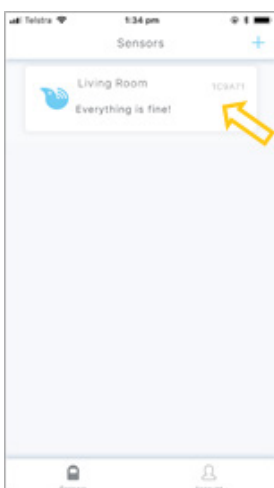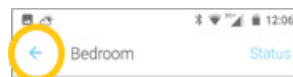

From the *My Sensors* page, select the sensor for which you want to set alert sensitivity.

The *Sensor* graph page will open.



This guide is for Android 5+ Smartphone users.



AbiBird

## Step 2: Pause notifications



The *Sensor* graph page has the name of the sensor at the top of the page.

From the panel at the bottom of the *Sensor* graph page, select **Pause**. The **Pause** icon changes state to **Resume**.



## That's it



The sensor's notifications are paused. While AbiBird will continue updating the graph every hour, all notifications of unusual activity from this sensor will stop.

Working in the app, you have:

- Accessed the sensor's graph page and paused notifications.

### IMPORTANT

Remember to press **Resume** – to 'un-pause' the sensor's notifications – when you are ready to receive notifications from the sensor again (when your cared-for person is back home).

## Related Guides

*How to Set or Change Notification Preferences*